CALEX® Cap Device

For Research Use Only.
Not for use in diagnostic procedures.

B-CALEX-C50-U

B-CALEX-C200-U

B-CALEX-C500-U

Revision: 2016-10-31

INSTRUCTION FOR USE

ENGLISH

INTENDED USE

The BÜHLMANN CALEX® Cap is a single use tube filled with extraction buffer. It is intended for sampling of human stool specimens and extraction of calprotectin for subsequent laboratory analysis with BÜHLMANN calprotectin assays.

This product is for research use only. It is not intended for use in diagnostic procedures.

DEVICE

Device	Content	Code	Preparation	Storage/Shelf life
CALEX® Cap	50	B-CALEX-C50-U	Ready to use	2-8 °C until the expiration date printed on the labels
Device	200	B-CALEX-C200-U		
filled with 5 mL extraction buffer	500	B-CALEX-C500-U		

SAFETY PRECAUTIONS

- Read the instruction carefully before you use the CALEX® Cap device.
- The extraction buffer does not contain components of human origin.
- The devices must be equilibrated at room temperature (18-28°C) before starting the extraction.
- The devices must not be used after the expiry date printed on the labels.
- The devices can be shipped at ambient temperature.
- The devices must not be reused for stool sample extraction.

HOW TO PROCEED WITH DIFFICULT STOOL SAMPLES

- **Liquid stool samples** can be pipetted directly into the CALEX® Cap device. Unscrew the blue cap and pipet 10 µL of stool sample into the device. Recap the CALEX® Cap device and proceed with step 4 on page 4.
- **Stool that does not stick to the grooves** can be sampled as follows: Position the stool sample on the wall of your stool collection tube or place it on a spatula/spoon. The dosing tip (refer to Fig. B, page 3) has to be pressed along the wall of the tube or against the surface of the spatula/spoon. Rotate dosing tip until the grooves are completely filled with stool. Continue with step 3 on page 4.

Components of the CALEX® Cap

Fig. A:

CALEX® Cap at delivery

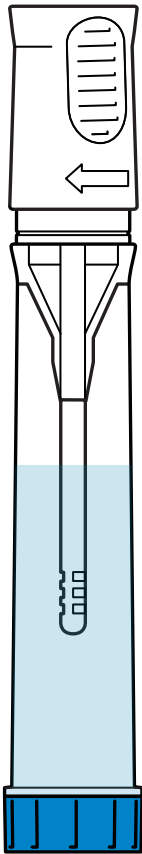
Fig. B:

Sampling pin


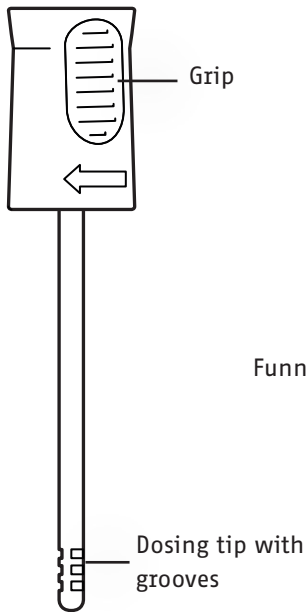
Fig C:

CALEX® Cap body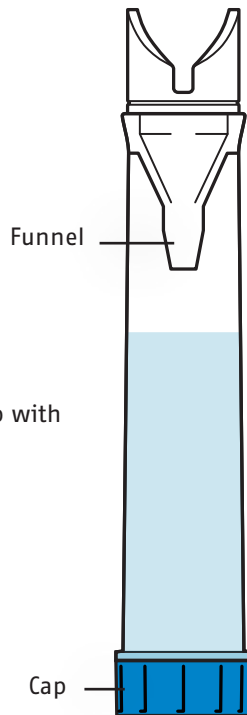


INSTRUCTION FOR USE

1. Hold the CALEX® Cap device upright and remove the **white** sampling pin by simultaneously turning and pulling it upwards (picture A).

Do not unscrew the blue cap!


A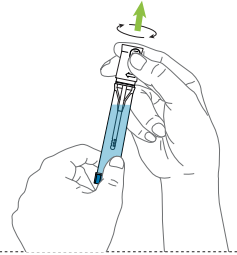

2. Dip the sampling pin with the grooves into the stool, turn the pin and remove it afterwards. Repeat the procedure at 3-5 different positions of the sample in order to fill the grooves completely (picture B).


Excess stool is stripped off in the funnel during re-introduction of the sampling pin into the CALEX® body.

B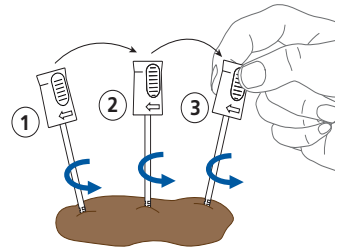


3. Re-introduce the sampling pin into the CALEX® body and push it back into the final locking position (you feel and hear two clicks, picture C).

C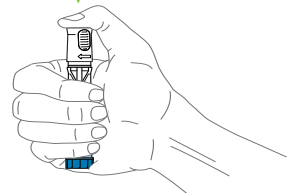


4. Vortex the CALEX® Cap device with the white head down vigorously on a vortex mixer for 30 sec (picture D) and allow it to stand for 10 min on the white head. If the stool does not completely come off the grooves, repeat the procedure until the grooves are completely free of stool. That is easily monitored by checking that the grooves are empty (refer to page 3, fig. B) before you proceed with step 5.

D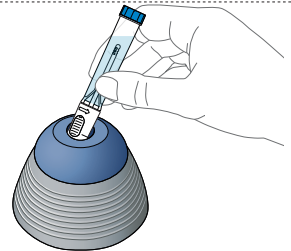

5. The stool sample is now diluted 1:500. Unscrew the blue cap (picture E) to access the sample for further analysis.

E

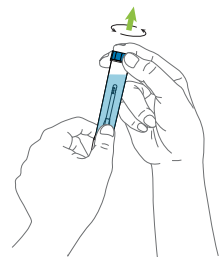6. Tightly recap the CALEX® Cap device after usage.

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CALEX® is a registered trademark of BÜHLMANN in many countries of the world