

ScheBo[®] Quick-Prep[™]

Faecal sample preparation kit


**For Research Use Only.
Not for use in diagnostic procedures.**

B-CAL-S050-U 50 tubes

Version 2015-10-02

Contents

50 tubes for the extraction of human Calprotectin from stool samples to be measured with BÜHLMANN kits; for professional use only. For research use only. Not for use in diagnostic procedures.

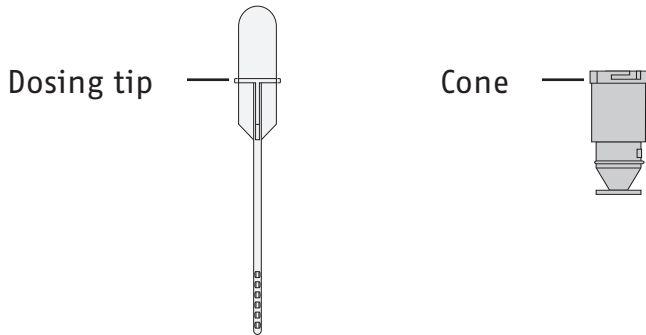


Ready to use: Filled with 1.3 ml of extraction buffer, store at 2-8 °C until the expiration date.

Preparation of sample




Turn the dosing tip counter-clockwise and remove it by pulling it upwards.

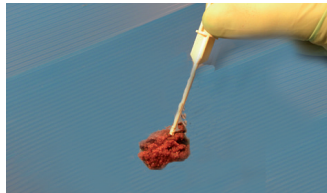

Insert the dosing tip into the fecal sample (depth appr. 2 cm). Repeat sampling 3-5 times in order to fill the grooves completely. Insert the dosing tip back into the tube and firmly close it by turning the dosing tip clockwise.



Homogenize the sample by vortexing for 30 sec., let it stand for 10 min. and vortex again for 30 sec. using a vibration mixer. Repeat this step until the faeces has completely been removed from the dosing tip.

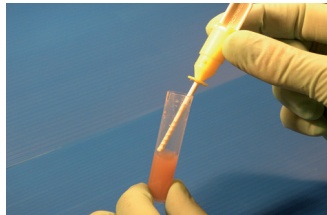


Turn the cone containing the dosing tip counter-clockwise and remove by pulling it upwards.



Extract processing and dilution for the Calprotectin determination are described in the instruction for use of the respective kit.

Extraction device

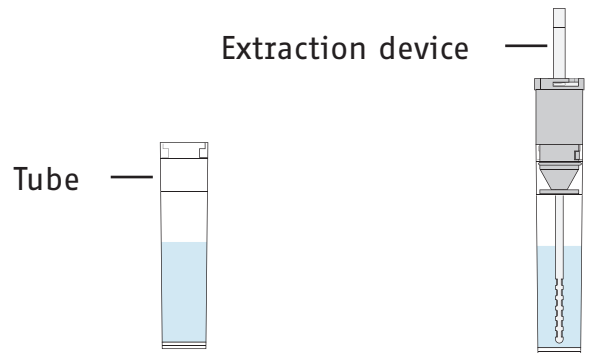

Symbol



Explanation

Use by
Verwendbar bis
Utiliser jusqu'au
Utilizzare entro
Fecha de caducidad



Consult instructions for use
Gebrauchsanweisung beachten
Consulter le mode d'emploi
Consultare le istruzioni per l'uso
Consulte las instrucciones de uso



Temperature limitation
Zulässiger Temperaturbereich
Limites de température
Limiti di temperatura
Limite de temperatura



Batch code
Chargenbezeichnung
Code du lot
Codice del lotto
Codigo de lote



Do not re-use
Nicht wiederverwendbar
Ne pas reutiliser
Monouso
No reutiliza

 **BÜHLMANN**

BÜHLMANN Laboratories AG
Baselstrasse 55
CH-4124 Schönenbuch/Basel
Switzerland

Phone +41 61 487 12 12
Fax orders +41 61 487 12 99
info@buhlmannlabs.ch
www.buhlmannlabs.ch