

On board stability: at least 60 days at up to 15 °C.

Shelf life of opened reagents at 2-8 °C: see expiry date on package label.

Do not freeze reagents! Shelf life of unopened reagents at 2-8 °C: see expiry date on package label.

#### Reagent Storage and Stability

Table 1

Reagents	Quantity	Code	Preparation
Reaction Buffer (R1)	1 vial 35 mL	B-KCAL-R1-U	Ready to use
Immunoparticles (R2)	1 vial 7 mL	B-KCAL-R2-U	Ready to use

#### Reagents Supplied

The BÜHLMANN fCAL® turbo test is a particle enhanced turbidimetric immunoassay (PETIA) and allows quantification of calprotectin in fecal extracts on clinical chemistry analyzers. Fecal samples are extracted with extraction buffer (B-CAL-EX-U) using the BÜHLMANN CALEX® Cap extraction device (Code: B-CALEX-C50-U / B-CALEX-C200-U / B-CALEX-C500-U) or a manual weighing method and applied at a final dilution of 1:500. The extracts are incubated with reaction buffer and mixed with polystyrene nanoparticles coated with calprotectin-specific antibodies (immunoparticles). Calprotectin available in the sample mediates immunoparticle agglutination. Sample turbidity, measured by light absorbance, increases with calprotectin-immunoparticle complex formation and is proportional to calprotectin concentration. The detected light absorbance allows quantification of calprotectin concentration via interpolation on an established calibration curve.

#### Principle of the Assay

followed by an additional 1:10 dilution in BÜHLMANN extraction buffer.

**Alternative method:** Manual weighing 1:50 with BÜHLMANN extraction buffer (Code: B-CAL-EX12-U), centrifugation (10 min, at 1000 x g)

minutes at 1000 x g and continue with the assay procedure.

**Important:** After extraction, centrifuge the CALEX® Cap Device for 10 minutes at 1000 x g and continue with the assay procedure.

The BÜHLMANN fCAL® turbo is designed for calprotectin quantification in fecal sample extracts. Fecal samples are collected, extracted and diluted to a final concentration of 1:500 using the BÜHLMANN CALEX® Cap Device (Code: B-CALEX-C5-U / B-CALEX-C200-U / B-CALEX-C500-U).

#### Specimen Collection and Handling

Table 2

Reagents	Quantity	Code
BÜHLMANN fCAL® turbo Calibrator Kit	1 x 6 vials 1 mL/vial	B-KCAL-CASET-U
BÜHLMANN fCAL® turbo Control Kit	3 x 2 vials 1 mL/vial	B-KCAL-CONSET-U
CALEX® Cap Device	50 tubes 200 tubes	B-CALEX-C50-U B-CALEX-C200-U
Extraction Kit	12 bottles 125 mL	B-CAL-EX12-U

#### Materials required but not provided

- This test is for *in vitro* use only, and must be handled by qualified personnel, in accordance with good laboratory practices (GLP).
- The immunoparticles contain potentially infectious substances of animal origin and should be handled with due caution. Disposal of any discarded materials should be in accordance with local requirements.
- R1 contains MOPS that can be irritating to eye and skin. Handle with due caution.
- R2 contains polystyrene nanoparticles.

#### Warnings and Precautions







#### Shipping damage

Please notify your distributor, if this product was received damaged

#### REACH

None of the materials and reagents in the kit require a Safety Data Sheet (MSDS) according to CLP-Regulation (EC) No 1272/2008 and directive EC 1907/2006 (REACH).

#### Symbols Key

	Manufacturer		Catalogue Number
	Expiration date		Lot number
	Consult Instructions for Use		Temperature limitations

#### Manufacturer

BÜHLMANN Laboratories AG  
Baselstrasse 55  
4124 Schönenbuch, Switzerland



## BÜHLMANN fCAL® turbo

Calprotectin turbidimetric assay  
for professional use

### Reagent Kit

B-KCAL-RSET-U

This product is for research use only  
It is not intended for use in diagnostic procedures

#### BÜHLMANN Laboratories AG

Baselstrasse 55  
4124 Schönenbuch  
Switzerland  
Tel.: +41 61 487 1212  
Fax: +41 61 487 1234  
info@buhlmannlabs.ch

EN: The complete Instruction for Use can  
be downloaded on  
[www.buhlmannlabs.ch](http://www.buhlmannlabs.ch)

#### Intended Use

The BÜHLMANN fCAL® turbo is a test for the quantitative determination of calprotectin in human stool specimens.

This product is for research use only. It is not intended for use in diagnostic procedures.

**Specimen storage:** Fecal samples can be stored at 2-8 °C for 6 days. The extracts are stable at 2-8 °C for up to 6 days and can be stored frozen (at -20 °C) for at least 6 months.

## Assay Procedure

### Application Notes / Assay Installation

Assay procedures for the BÜHLMANN fCAL® turbo are established on several clinical chemistry analysers. Validated Application Notes describing installation and analysis on specific instruments are available from BÜHLMANN upon request. Corresponding instrument manuals must be considered for instrument setup, maintenance, operation and precautions.

### Reagent Preparation

The reagents supplied are ready to use. Mix gently before loading onto the instrument. The reagent bottles may fit directly into the instrument, unless otherwise stated in the Application Note.

### Establishment of the Calibration Curve

The BÜHLMANN fCAL® turbo Calibrator (Code: B-KCAL-CASET-U) is used to establish a six point standard curve according to the instrument manual. Calibrator values are lot-specific. A new calibration must be performed for each new calibrator lot. Otherwise, calibration should be performed every 4 to 8 weeks according to the instrument specific Application Notes. Refer to the QC-data sheet provided with the BÜHLMANN fCAL® turbo Calibrator for assigned calibrator values. Contact BÜHLMANN support if calibration cannot be performed without error.

### QC Controls

The BÜHLMANN fCAL® turbo Controls, Low and High, (Code: B-KCAL-CONSET-U) must be assayed each day before running fecal sample extracts to validate the calibration curve. The controls have assigned value ranges indicated on the QC-data sheet supplied with each lot of the BÜHLMANN fCAL® turbo Control Kit. The control measurements must be within the indicated value ranges to obtain valid results for fecal sample extracts.

## Performance Characteristics

The presented performance characteristics have been determined on Mindray BS-380 (registered trademark of Mindray Medical International).

**Detection Limits:** BÜHLMANN fCAL® turbo has a Limit of Quantification (LoQ) of 19 µg/g calprotectin on the BS-380 analyser, where LoQ is defined as the lowest actual amount of an analyte that can be reliably detected and at which the total error meets accuracy requirements.

**Linearity:** BÜHLMANN fCAL® turbo calibrated with the calibrator kit (B-KCAL-CASET-U) is linear within the range of 21-2058 µg/g on the BS-380 analyser. Concentrations outside this range will be diluted by the analyser-specific protocol 1:10 and retested.

**Precision:** The precision of BÜHLMANN fCAL® turbo calibrated with the calibrator kit (B-KCAL-CASET-U) was determined in a 20-day precision study to CLSI guideline EP5-A3. Four fecal extracts and 2 controls (B-KCAL-CONSET-U) were measured on the BS-380 analyser.

Fecal extracts	Mean value (µg/g)	Within-Run CV (%)	Between-Day CV (%)	Between-Run CV (%)	Total CV (%)
Low	52	3.91	3.10	3.60	6.15
Medium low	84	1.99	3.98	0.87	4.54
Medium high	418	0.64	0.15	1.54	1.67
High	2024	0.32	1.79	1.12	2.14
Control low	74	4.43	5.24	0.51	6.88
Control high	259	0.62	2.48	1.52	2.97

Table 3: Precision results

**Hook Effect:** For the BÜHLMANN fCAL® turbo no hook effect is observed for calprotectin concentrations up to 8000 µg/g.

**Interference:** No interference is detected with Haemoglobin (0.05 g/L), Triglycerides (Intralipid®) (1 g/L) or Bilirubin (0.5 mg/L) on the BS-380 analyser. As samples for BÜHLMANN fCAL® turbo are extracted at high dilution (1:500) interfering concentrations lower than standard were used. No interference was observed with standard, orally-administered drugs and food additives prescribed for gastrointestinal complaints. The interference study was designed in accordance with the CLSI guideline EP07-A2.

If the control values are not valid, perform the following procedures: 1) repeat QC control measurement with fresh controls. 2) recalibrate the instrument. Contact BÜHLMANN support if valid control values cannot be reproduced.

## Fecal Sample Extract Measurement

Once a calibration curve is established and validated with the QC controls, fecal extracts may be measured. Perform fecal extract measurement according to the Application Note and instrument manual.

## Results

The results are calculated automatically on the analyser and presented in µg/g.

## Limitations

- Sample carryover: a minor sample carryover (0.5 %) which might cause slightly increased levels has been detected.
- Reagent R2, occasionally frozen cannot be used anymore. Freezing R2 will lead to reduced sensitivity and precision in low level samples and in the worst case to decreased measurement levels.
- The assay is designed for fecal extract samples prepared using the specific BÜHLMANN extraction buffer (B-CAL-EX-U). Application of other extraction buffers could lead to incorrect results.
- Evaporation of calibrators and controls on the analyser could lead to incorrect results. Run the assay immediately after loading the analyser.
- A minor hook effect in extracts >8000 µg/g might occur. Values >8000 µg/g are rare (<0.5 %) but might be measured falsely between 1000 and 2000 µg/g. It has not been observed that the value of extracts >8000 µg/g could be determined as below 1000 µg/g.

**Method Comparison:** Results obtained with BÜHLMANN fCAL® turbo on the BS-380 analyser were compared with those obtained by the BÜHLMANN fCAL® ELISA by Passing-Bablok regression. The study included 112 fecal extracts in the range of 30-1800 µg/g.

	N	112
Linear Regression	Slope	1.05
	Coefficient of determination (R <sup>2</sup> )	0.91
Altman Bland analysis	Bias (%)	-2.5
	Lower LoA*	-47.8 %
	Upper LoA*	42.9 %

Table 4. Method comparison results.  
\*LoA =Limits of agreement

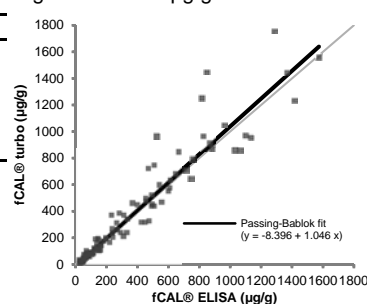


Figure 1. Scatterplot of 112 samples included in the method comparison analysis