

## INCIDENT REPORTING IN EU MEMBER STATES

If any serious incident in relation to this device has occurred, please report without delay to the manufacturer and competent authority of your Member State.

## SHIPPING DAMAGE

Please notify your distributor, if this product was received damaged.

## SYMBOLS

BÜHLMANN use symbols and signs listed and described in ISO 15223-

1. For definition of symbols see the symbol glossary at:

[www.buhmannlabs.ch/support/downloads/](http://www.buhmannlabs.ch/support/downloads/)

In addition the following symbols and signs are used:



**EN:** electronic instruction for use available in different languages at:/ **BG:** електронни инструкции за употреба на различни езици на адрес:/ **CS:** elektronický návod k použití dostupný v různých jazycích na adrese:/ **DA:** elektronisk brugsanvisning på forskellige sprog på:/ **DE:** elektronische Gebrauchsanweisung in verschiedenen

Sprachen verfügbar unter:/ **EL:** ηλεκτρονικές οδηγίες χρήσης διαθέσιμες σε διάφορες γλώσσες στη διεύθυνση:/ **ES:** instrucciones de uso electrónicas disponibles en diferentes idiomas en:/ **ET:** elektrooniline kasutusjuhend, mis on saadaval erinevates keeltes aadressil:/ **FR:** un mode d'emploi électronique disponible en différentes langues à l'adresse:/ **HU:** külnöböz nyelveken elérhető elektronikus használati utasítás a következő címen:/ **IT:** istruzioni elettroniche per l'uso disponibili in diverse lingue su:/ **LT:** elektroninės naudojimo instrukcijos įvairiomis kalbomis:/ **LV:** dažādās valodās pieejama elektroniska lietošanas instrukcija:/ **NO:** elektronisk instruksjon for bruk tilgjengelig på forskjellige språk på:/ **PL:** elektroniczna instrukcja obsługi dostępna w różnych językach na stronie:/ **PT:** instrução electrónica para utilização disponível em diferentes línguas em:/ **RO:** instrucțiuni electronice de utilizare disponibile în diferite limbi la adresa:/ **SK:** elektronický návod na použitie dostupný v rôznych jazykoch na:/ **SL:** elektronska navodila za uporabo so na voljo v različnih jezikih na:/ **SR:** elektronsko uputstvo za upotrebu dostupno na različitim jezicima na:/ **SV:** elektronisk bruksanvisning på olika språk på följande adress:

[www.buhmannlabs.ch/support/downloads/](http://www.buhmannlabs.ch/support/downloads/)

**US Distribution**  
**BÜHLMANN Diagnostics Corp**  
Amherst, NH 03031, USA  
Tel: (844)300-979  
[info@buhmannlabs.com](mailto:info@buhmannlabs.com)



## BÜHLMANN fCAL® turbo

Calprotectin turbidimetric assay  
for professional use

## Control Kit

B-KCAL-CONSET  
Version A4.2

For *In Vitro* Diagnostic Use

CLIA Complexity: High

Rx Only



**BÜHLMANN Laboratories AG**  
Baselstrasse 55  
4124 Schönenbuch  
Switzerland  
Tel.: +41 61 487 1212  
Fax: +41 61 487 1234  
[info@buhmannlabs.ch](mailto:info@buhmannlabs.ch)



[www.buhmannlabs.ch/support/downloads/](http://www.buhmannlabs.ch/support/downloads/)

## INTENDED USE

The BÜHLMANN fCAL® turbo Control Kit is intended for use with the BÜHLMANN fCAL® turbo Reagent Kit, for quality control, in the determination of fecal calprotectin levels in extracted stool sample.

For laboratory use only.

## CONTROL VALUE

Control values are assigned according to a value transfer protocol (ref. 1-2) and are indicated in the enclosed QC-data sheet. The control material comprises blood-derived human calprotectin and is standardized against internal reference material.

## REAGENTS SUPPLIED

Reagents	Quantity	Code	Preparation
<b>Controls Low / High</b> Controls containing an assigned concentration of human calprotectin	3 x 2 vials 1 mL/vial	B-KCAL-CONSET	Ready to use

Table 1

## REAGENT STORAGE AND STABILITY

Unopened controls
Store at 2-8 °C. Do not use kit past expiration date printed on the labels.
Opened controls
Store for up to 3 months at 2-8 °C, capped.

Table 2

## MATERIALS REQUIRED BUT NOT PROVIDED

Reagents	Quantity	Code
<b>BÜHLMANN fCAL® turbo Reagent Kit</b> Reaction Buffer (R1) Immunoparticles (R2)	1 vial/35 mL 1 vial/7 mL	B-KCAL-RSET B-KCAL-RSET-US <sup>1</sup>
<b>BÜHLMANN fCAL® turbo Calibrator Kit</b> Calibrators 1-6 for establishment of six point calibration curve	1 x 6 vials 1 mL/vial	B-KCAL-CASET

Table 3

<sup>1</sup> For US customers only

## WARNINGS AND PRECAUTIONS

- This test is for *in vitro* diagnostic use only.
- This kit contains components classified in accordance with the Regulation (EC) No. 1272/2008: 2-methyl-4-isothiazolin-3-one hydrochloride (conc. ≥ 0.0015%), thus the reagents may cause allergic skin reactions (H317).
- Before measuring please equilibrate reagents, controls, calibrators and samples as described in the application note.

- Do not mix controls of different lots or switch caps between reagents.
- Avoid evaporation of the controls.
- The controls contain components of human origin. Although tested and found negative for HBV, HCV and HIV, the controls should be handled as if capable of transmitting infections and should be handled in accordance with Good Laboratory Practices (GLP) using appropriate precautions. Disposal of any discarded materials should be in accordance with local requirements.

## ASSAY PROCEDURE

### Application notes/ assay installation

The assay procedure for the BÜHLMANN fCAL® turbo has been established on several clinical chemistry analyzers. Validated application notes describing installation and analysis on specific instruments are available from BÜHLMANN upon request.

### QC controls

The BÜHLMANN fCAL® turbo Control kit must be assayed each day before running patient fecal sample extracts. This is to validate the calibration curve established with the BÜHLMANN fCAL® turbo Calibrator kit. The controls have assigned, lot-specific value ranges indicated on the QC-data sheet enclosed. The control measurements must be within the indicated value ranges to obtain valid results for patient fecal sample extracts.

If the control values are not valid, repeat measurement with fresh controls. If control values remain invalid, recalibrate the instrument. If valid control values cannot be reproduced, after performing the steps described above, contact BÜHLMANN support.

## CHANGELOG

Date/Version	Change
2023-07-12/ A4.2	Global Instruction for use starting at version number A4.2. The US version A2-3 has been skipped during instruction for use harmonization. Revision of chapter Symbols.

## REFERENCES

1. Bliurup-Jensen et al.: Clin Chem Lab Med 2001; 39, 1110-22.
2. Bliurup-Jensen et al.: Clin Chem Lab Med 2008; 46, 1470-9.