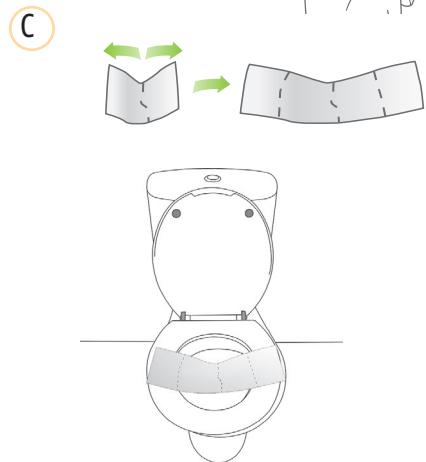
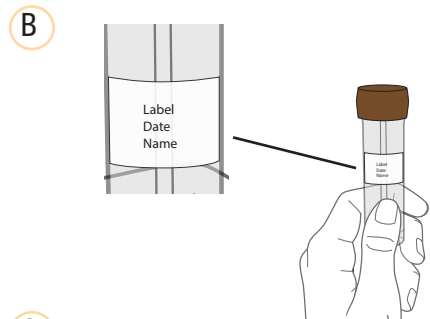
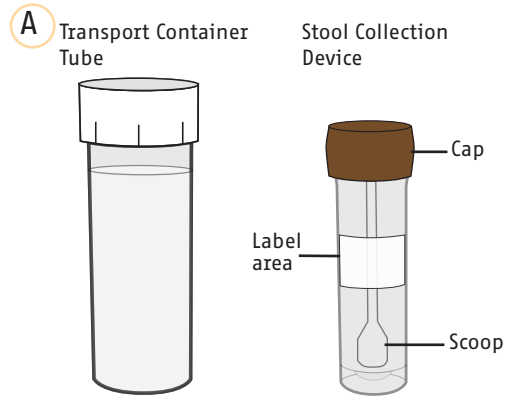
B-CAL-SAMPLER25, Rev.: 08-03-2016

ENGLISH

INTENDED USE

The BÜHLMANN Simple Sampler® Stool Collection Set is used to collect stool samples, limiting contamination and direct contact with the stool.

MATERIALS AND ACCESSORIES

Quantity	Components
1	Stool Collection Device
2	Stool Collection Sheets
2	Examination gloves (Nitril)
1	Instruction for Use
1	Transport Container Tube

SAFETY PRECAUTIONS AND IMPORTANT ADVICE

- Label the Stool Collection Device before starting to collect a stool sample (B).
- Use disposable gloves to avoid contamination.
- Do not put stool in the Transport Container Tube.
- Flush Stool Collection Sheets in the toilet after use.
- The device can be shipped at ambient temperature.
- The device must not be reused.

PROCEDURE

1. Empty your bladder first if necessary, as urine can affect the sample.
2. Label the Stool Collection Device with date and name (B). Unfold the Stool Collection Sheet by holding the open ends and gently pulling outwards (C).
3. Place the Stool Collection Sheet on top of the toilet seat in the rear (C).

CAUTION: The sheet must not touch the toilet water.

4. Ensure the Stool Collection Sheet is fixed properly on the toilet seat.

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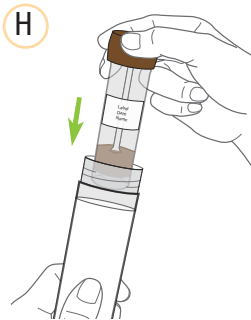
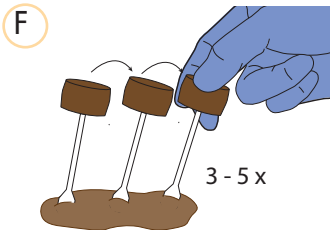
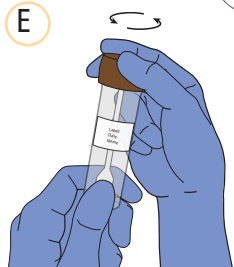
ENGLISH

- 5 Make sure that your stool sample is caught on the Stool Collection Sheet (D).
- 6 Put on examination gloves.
- 7 The stool sample is collected with the Stool Collection Device (A). Open the device and use the small scoop (A) to collect stool material.
- 8 Dip the scoop (A) of the Stool Collection Device into the stool (F). Fill the scoop completely with stool. Repeat this procedure 3 to 5 times in order to fill the Stool Collection Device half full.

CAUTION: Make sure to put stool into the Stool Collection Device and do not fill it more than half full.

- 9 Place the scoop (A) back into the Stool Collection Device and close it (G).
- 10 Remove gloves from both hands and dispose them properly. Wash hands to prevent contamination.
- 11 Put the Stool Collection Device into the Transport Container Tube (H) and close it.

NOTE: If Stool Collection Device shipment is necessary, make sure it is safely stored in the Transport Container Tube and ask for suitable UN3373 packaging.

NOTE: If you can't return the Stool Collection Device the same day, please keep the packaged sample in your fridge.

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