

BÜHLMANN fCAL[®] ELISA


Procedure EK-CAL-U

Extraction Methods

General comments

Homogenize well. Fibrous, insoluble material occasionally present in the sample and turbidity of the final extract do not affect results.

1. Manual extraction procedure

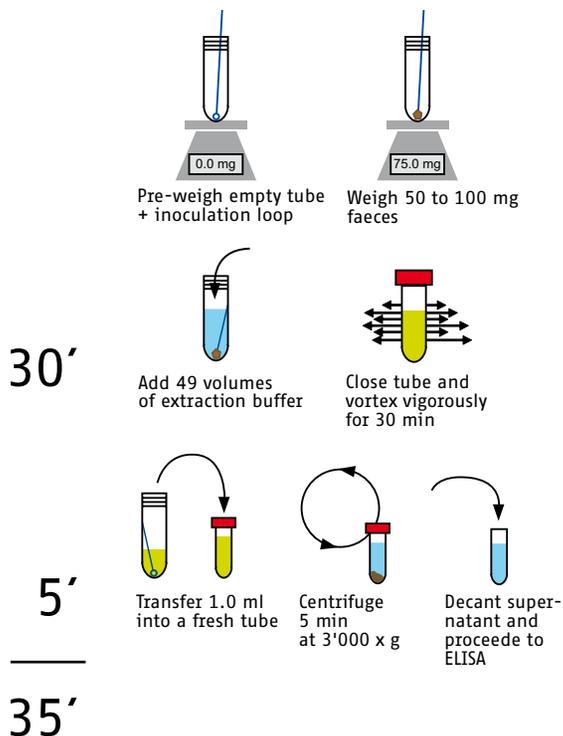

2. Extraction Devices

15'

CALEX[®] Cap

- Remove the white sampling pin from the tube by pulling it upwards.
- Insert the sampling pin 3-5 times into the sample and make sure that grooves are completely filled.
- Re-introduce the sampling pin into the CALEX[®] Cap tube and push it back into the final locking position.
- Vortex the CALEX[®] Cap device with the white head down vigorously on a vortex mixer for 30 sec and allow it to stand for 10 min on the white head. Repeat the procedure until the grooves are completely free of stool.
- Centrifuge it for 5 minutes at 500 x g and continue with the ELISA.

6'

Smart-Prep

- Press the sample into the hollow cavity in the base cap and level off the surface.
- Firmly press the tube onto the base cap and detach the tag.
- Add 4 mL of extraction buffer.
- Homogenize the sample for 1 min using a vibration mixer.
- Pour the extract into a fresh tube.
- Centrifuge it for 5 minutes at 3'000 x g and continue with the ELISA.

ELISA

All reagents must be adjusted to room temperature

Precoated Microtiter Plate

↓ ↻ wash 2x

100 µL Calibrators, Controls or diluted Samples (1:50 or 1:150)

30' ↓ ↻ incubate 30 min at 18-28°C on a plate rotator

wash 3 x

add 100 µL Enzyme Label

30' ↓ ↻ incubate 30 min at 18-28°C on a plate rotator

wash 5 x

add 100 µL TMB Substrate

15' ↓ ↻ incubate 15 min at 18-28°C on a plate rotator

add 100 µL Stop Solution

→ Read at 450 nm

75'

Time to Result: <2 hours



BÜHLMANN fCAL® ELISA

Characteristics EK-CAL-U

Intended Use

Extraction and quantitative in vitro determination of human Calprotectin in stool samples. For Research Use Only. Not for use in diagnostics procedures in the US.

Pre-Analytics

- Sample required: 50-100 mg human faeces
- Sample collection: plain tubes without chemical or biological additives
- Sample storage: at 2-8°C up to 6 days
- Storage of extracts: at 2-8°C up to 7 days and at ≤-20°C at least 24 months

Special Equipment

- Multi tube vortex mixer
- Precision balance (10-150 mg)
- Fecal extraction device (to be ordered separately)
- Micro centrifuge

Calibrators & Controls

- 5 Calibrators* lower range ELISA procedure: 10-600 µg/g
extended range ELISA procedure: 30-1800 µg/g
- 2 Controls low, high

*** The standard sample dilution has already been factored in the calibrators!**

Assay Performance Data

For lower and extended range ELISA procedure

Intra-assay precision 4.7 %

n=6; range: 52.5-1246 µg/g
20 duplicates; CV range: 2.7-8.1 %

Inter-assay precision <15 %

n=10; range: 18.1-1764 µg/g
20 repetitions; CV range: 6.6-14.5 %

Dilution linearity 103 %

12 samples diluted 1:50 up to 1:12800
recovery range: 82-124 %

Spiking recovery 100 %

2 samples spiked with native Calprotectin from human serum; recovery range: 97-103 %

Analytical sensitivity <10 µg/g*1

n=20 each, in duplicates; mean + 2 SD <30 µg/g*2

Functional sensitivity <10 µg/g*1

n=28; range: 5.2-2080 µg/g <30 µg/g*2
Each 20 x in duplicates; limit of acceptance 15 % CV


Precision profile <15 %

The precision profile allows a precise measurement between 10-600 and 30-1800 µg/g.

*1 lower range ELISA procedure

*2 extended range ELISA procedure

Typical Standard Curve (4 Parameter Logistics)

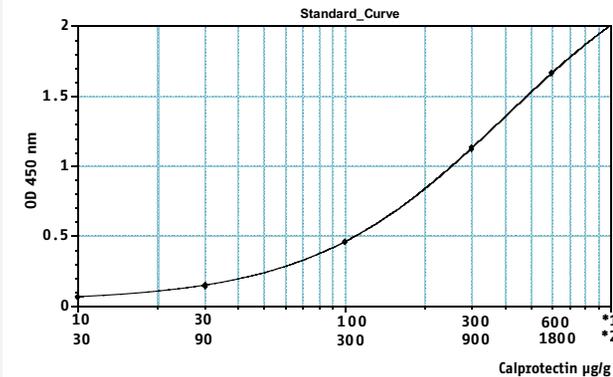

Available For Research
Use Only in the US.

Ordering code:

- EK-CAL-U 96 wells
- EK-CAL2-U 2 x 96 wells
- EK-CAL2-WEX-U 2 x 96 wells

Other products:

- EK-MRP8/14-U 96 wells

US: BÜHLMANN fCAL® ELISA (EK-CAL-U, EK-CAL2-U, EK-CAL-WEX-U) and MRP8/14 ELISA (EK-MRP8/14-U) are available For Research Use Only in the US.

Canada: BÜHLMANN fCAL® ELISA (EK-CAL) is Health Canada Licensed (No. 80726).



BÜHLMANN Laboratories AG
Switzerland
info@buhlmannlabs.ch
www.buhlmannlabs.ch
Phone +41 61 487 12 12

BÜHLMANN Diagnostics Corp
USA / Canada
info@buhlmannlabs.com
www.buhlmannlabs.com
Phone 844 300 9799