

# BÜHLMANN fCAL<sup>®</sup> turbo

Immuno turbidimetric fecal CALPROTECTIN Assay

Unique in  
Speed  
Quality  
Flexibility



## Fecal Calprotectin result within 10 minutes

- Streamline your workflow
- Integrate fecal samples
- Use random access and save hands-on time
- Measuring Range from 20 to 8000 µg/g

## Simplify and improve fecal extraction with CALEX<sup>®</sup> Cap

- Prefilled CALEX<sup>®</sup> extraction device
- High quality for fast and efficient extraction
- Total analysis time (TAT) of 30 minutes
- Compatible for Total Laboratory Automation solutions

BÜHLMANN fCAL<sup>®</sup> turbo and CALEX<sup>®</sup> Cap are available for Research Use Only in the US. Not for use in diagnostic procedures.

## Fecal Calprotectin result within 10 minutes

**BÜHLMANN fCAL® turbo, the turbidimetric immunoassay, is a flexible solution to be applied on most clinical chemistry analyzers.**

The technology is a milestone in fecal calprotectin quantification. It allows very rapid and flexible random access use, as well as being the ideal solution for high throughput applications in the research laboratory. The fCAL turbo reduces the hands-on time dramatically and allows to report calprotectin results from human stool samples within very short time.

Centrifuged fecal extracts derived from the unique BÜHLMANN extraction device CALEX® Cap can be applied directly on your existing analyzer. No special equipment is required. BÜHLMANN continuously works on the list of applications. The analysis itself takes ten minutes and covers a range from 20 to 8000 µg/g. Re-testing of very high concentrated samples might be done automatically.

Available application notes	Siemens	Advia 1650/1800, Advia 2400, Advia XPT
	Roche	cobas® 6000/8000 (c501/2 + c701/2), Integra800*, Integra400+, c111
	Abbott	Architect (c series), Alinity (planned)
	Beckman	AU series, DxC
	Mindray	BS-380, BS-200E, BS-400, BS-240*
	Thermo Fisher	Konelab, Indiko
	Ortho	Vitros 5600
	The Binding Site	Optilite
	Horiba	ABX Pentra 400
	IDS	iSYS
	Biosystems	BA200*

\*under validation

### PRODUCT DESCRIPTION

Method	Particle-enhanced turbidimetric immunoassay (PETIA)
Sample Type	Fecal extracts
Kit Format	2 reagents (wedge bottles for direct loading on many platforms) Reagents last for ~200 tests (platform dependent)
Sample Preparation	CALEX® Cap extracts can be used directly without additional dilution
Reagent on board	Stable for at least 60 days
Calibration	Stable for up to 60 days
Calibration range	0-2000 µg/g
Measuring range	20-8000 µg/g
Sample volume	~10 µL centrifuged fecal Extract (1:500)
Time to result	~10 min CALEX® Extraction ~20 min

## Fecal extracts on Clin Chem Platforms

Centrifuged extracts are free from undigested diet fibers. The bacterial load is reduced by ~95% and results in a comparable level as known from urine samples from subjects with UTI.

Carry-over effects in the high through-put analyzers is minimal and has been determined as <0.5%. Serum contaminations can be excluded.

Turbidimetric reagents are free of critical compounds. Interactions to other assays are highly unlikely and have not been observed.

## Simplify and improve fecal extraction with CALEX® Cap

The CALEX® Cap is a unique device for the fast and efficient quantitative extraction of calprotectin in stool specimen.

The prefilled tubes are ready to use. Three simple steps are required for extraction:

**STEP 1:** Dip the dosing tip into the fecal sample and fill the grooves completely with the matrix material.

**STEP 2:** Place the pin back into the tube, close tightly and extract by soaking and vortexing.


**STEP 3:** Centrifuge the extract within the CALEX® tube for 10 min.

The diluted extract (1:500) is ready to use in the turbidimetric assay. The CALEX® extraction is highly correlating to the manual weighting method.

## Use the established reference

The BÜHLMANN fCAL® turbo standardization is based on the well established BÜHLMANN fCAL® ELISA.

BÜHLMANN has given proof in keeping a constant and stable standardization of calprotectin. Our long experience with the complex protein and our large network in science will guarantee a continuous quality of the assay.

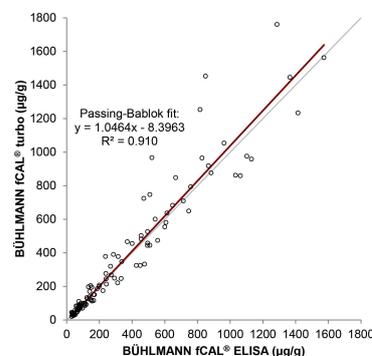

Method comparison BÜHLMANN fCAL® ELISA vs BÜHLMANN fCAL® turbo; n = 112

### BÜHLMANN fCAL® turbo Ordering Codes:

Kit	KK-CAL-U
Reagent Kit (~200 tests)	B-KCAL-RSET-U R1 35 ml, R2 7 ml
Calibrator Kit	B-KCAL-CASET-U 6 levels, 1 ml each; Ready to use
Control Kit	B-KCAL-CONSET-U 3 x 2 levels, 1 ml each; Ready to use

BÜHLMANN fCAL® and CALEX® are registered trademarks of BÜHLMANN in many countries.



BÜHLMANN Laboratories AG  
Switzerland  
info@buhlmannlabs.ch  
www.buhlmannlabs.ch  
Phone +41 61 487 12 12

BÜHLMANN Diagnostics Corp  
USA / Canada  
info@buhlmannlabs.com  
www.buhlmannlabs.com  
Phone 844 300 9799